



4-Hole Connector Receiver

1225 & 1215T

Instructions For Use



1225A



1225S/T



1225T-B



1215T

1. Application & Description

- This adapter is exclusively for lower extremity prosthetic fittings
- The 1225 is a Rotating Pyramid with hole
- The 1215T is Rotating Pyramid without hole
- **1225A is strictly for K2 ONLY.**
- Compatible with any 36mm threaded base, connecting a pyramid receiver system to a 36mm threaded receiver

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1225A	Aluminium	11mm	56g	158kg/350lbs
1225S	Stainless Steel	11mm	68g	136kg/300lbs
1225T	Titanium	11mm	39g	170kg/375lbs
1225T-B	Titanium	11mm	39g	170kg/375lbs
1215T	Titanium	11mm	42g	170kg/375lbs

- **Thread Locker:** Apply Loctite once alignment and connection are finalized

4. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F)
- **Storage:** Store at room temperature; avoid wet; very hot or extremely cold environments.
- **Warranty:** 2 years, provided it is used as intended without modification and only the original user.
- Do not reuse on another prosthesis
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



1225_1215T IFU REV. A (4-30-26)
Specifications subject to change.