



Rotating Receiver

Instructions For Use

1216, 1702T, & 1703T



1216A



1216S



1216T/T-W



1216T-B



1702T



1703T

1. Application & Description

- This adapter is exclusively for lower extremity prosthetic fittings
- It connects any pyramid system to a 36mm threaded receiver
- The four set screws anchor the mating system and allow for angular alignment
- **1216A is Strictly for K2 ONLY.**

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1216A	Aluminum	16.5mm	67g	158kg/350lbs
1216S	Stainless Steel	14mm	68g	136kg/300lbs
1216T/T-W	Titanium	14mm	48g	170kg/375lbs
1216T-B	Titanium	14mm	48g	170kg/375lbs
1702T	Titanium	45mm	71g	170kg/375lbs
1703T	Titanium	60mm	85g	170kg/375lbs

- **Recommended Torque:** Always cross tighten alignment set screws
- **M8 Alignment Set Screws:**
 - (Titanium/ Stainless Steel): **10-15Nm**
 - (Aluminum): **7-10Nm**
- **Thread Locker:** Apply Loctite once alignment and connection are finalized

3. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F)
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids.
- **Storage:** Store at room temperature; avoid wet; very hot or extremely cold environments.
- **Warranty:** 2 years, provided it is used as intended without modification and only the original user.
- Do not reuse on another prosthesis
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



1216_1702T_1703T IFU REV.A (4-30-2026)
Specifications subject to change.