



4-Hole Connector, Pyramid (With & Without Hole)

Instructions For Use

1217 & 1227T



1217A



1217S



1217T



1217T-B



1227T

1. Application & Description

- This adapter is exclusively for lower extremity prosthetic fittings.
- It connects any pyramid system to a 4-Hole adapter.
- The 1217 4-Hole Pyramid includes a through hole in the pyramid, while the 1227T does not.

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1217A	Aluminium	11mm	81g	158kg/350lbs
1217S	Stainless Steel	8mm	111g	136kg/300lbs
1217T	Titanium	8mm	81g	170kg/375lbs
1217T-B	Titanium	8mm	73g	170kg/375lbs
1227T	Titanium	8mm	73g	170kg/375lbs

- **Recommended Torque:** Always cross tighten Alignment Set Screws
- **M8 Alignment Set Screw:** 10-15Nm
- **M6 Mounting Screw:** 7-10Nm
- **Thread Locker:** Apply Loctite once alignment and connection are finalized.

3. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F)
- **Storage:** Store at room temperature; avoid wet; very hot or extremely cold environments.
- **Warranty:** 2 years, provided it is used as intended without modification and only the original user.
- Do not reuse on another prosthesis
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



1217_1227 IFU REV. A (4-30-2026)
Specifications subject to change.



MDSS GmbH

Schiffgraben 41 | 30175 Hannover, Germany