



4-Prong Rotating Base with Rotating Receiver, Ti

Instructions For Use

1257T



1257T

1. Application & Description

- This adapter is exclusively for lower extremity prosthetic fittings.
- The base is integrated into a laminated socket to connect a pyramid providing rotation with the **1223T** 4-Prong Rotating base, and **1216T** rotating receiver.
- **Compatibility:** The rotating receiver adapter is suitable for socket fittings for transtibial or small transfemoral amputees.

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1257T	Titanium	29mm	112g	170kg/375lbs

- **Recommended Torque:** Always cross tighten Alignment Set Screws
- **M8 Alignment Set Screw:**
 - (1216T): **10-15 Nm**
- **M6 Cap Screw:**
 - (1223T): **6-10 Nm**
- **Thread Locker:** Apply Loctite once alignment and connection are finalized.

3. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F). 
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids.
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments.
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments.
- **Warranty:** 2 years, provided it is used as intended without modification, and only for the original intended user.
- Do not reuse on another prosthesis. 
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.

4. Reinforcement Recommendations

- The clinician is responsible for determining the adequate layup and reinforcement based on the specific patient's usage, activity level, and expected loading.

1257T IFU REV.G (5-12-2026)

Specifications subject to change.