




i-01

Instructions for 1110 Dynamic Step Foot

Applications and product description for Dynamic Step Foot


1. Assembling and Fitting

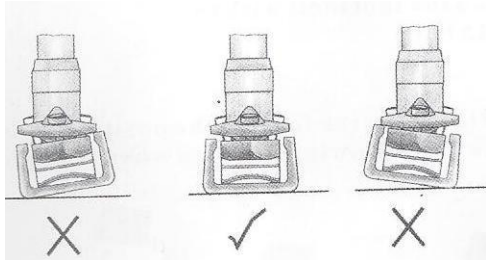

It is essential that the full width of the spring modules is parallel to the floor

2. Specification


1. When assembling, please cut the Teflon like below

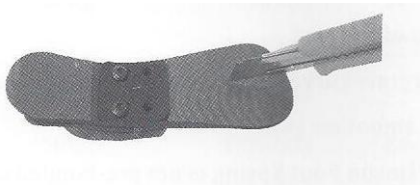

2. Please put on the carbon fiber stock to reduce the friction between the spring and footshell.

3. Please assemble the spring and footshell with two M5 bolt (8Nm torque).

4. Please put the foam in the position shown as the following pictures when assembling.

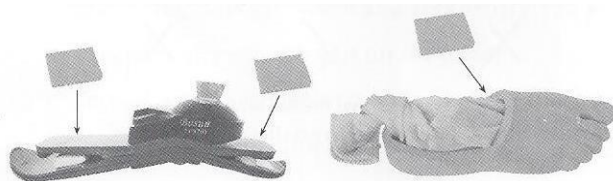



ST&G USA Corporation

www.stngco.com e-mail: info@stngco.com

2691 Saturn Street, Brea, CA 92821, USA
Tel : 1-714-524-0663 Fax : 1-714-364-8113

Room No. 609, 43, Yangsan-ro,
Yeongdeungpo-gu,
SEOUL, SOUTH KOREA
Tel : 82-2-6267-8195 Fax : 82-2-6267-8199

i-01

LOT	
Weight	
Date of Fitting	

Environment:

Store at room temperature, and avoid extremely hot, cold, or moist storage environments Storage

Temperature Range: between temperatures of -10°C to 50°C [14°F and 122°F]

Do not store in wet or very hot environment.

Allowable environmental conditions



Use – relative humidity: 0 % to 90 %, non-condensing

Storage/transportation – relative humidity: 20 % to 90 %, non-condensing

Splash - Fresh water, rain, salt water, urine, dust, sand, particles of foam cosmetics

cleaning required after contact with salt-laden air, salt water, chlorinated water, urine, sand

Unallowable environmental conditions

Storage/transportation: high vibrations, impacts

Cleaning agents containing solvents

CE Conformity

This product meets the requirements Council EU 2017/745 (EU) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future use.

Useful Life

Service life of the product is covered by the warranty period.

Disposal

The device and its packaging must be disposed of in accordance with respective national/local environmental regulations.

1110 IFU REV. E (07-12-24)

MD Medical Device



CORPORATE OFFICE
2691 Saturn Street | Brea, CA 92821
Tel: (714) 524-0663 | Fax: (714) 364-8113


ST&G CORPORATION
Rm. 911 43, Yangsan-ro
Yeongdeungpo-gu, Seoul, Korea


EC REP
MDSS GmbH
Schiffgraben 41 | 30175 Hannover, Germany