



4-Hole Connector Receiver

1218

Instructions For Use



1218A



1218S



1218T

1. Application & Description

- The 1218 4-Hole Connector Receiver is designed exclusively for prosthetic fittings of lower extremity amputations.
- It is suitable for both Transtibial and Transfemoral sockets where a 4-Hole to pyramid adapter is needed.

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1218A	Aluminium	19mm	84g	158kg/350lbs
1218S	Stainless Steel	16mm	160g	136kg/300lbs
1218T	Titanium	16mm	53g	170kg/375lbs

- **M6 4-Hole Mounting Screw Torque: 10Nm**
- **M8 Alignment Screw Torque:**
 - (Titanium/Stainless Steel): **15Nm**
 - (Aluminum): **10Nm**
- **Thread Locker:** Apply Loctite once alignment and connection are finalized

3. Installation Overview

- **Attachment:** The plate is anchored to the prosthetic socket using a 4-hole connector system and four socket mounting screws.
- **Compatibility:** Connects any modular component featuring a 4-hole threaded system to a pyramid system (such as a knee or foot).
- **Fastening:** Tighten all screws that are side by side using a torque wrench

4. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F)
- **Storage:** Store at room temperature; avoid wet; very hot or extremely cold environments.
- **Warranty:** 2 years, provided it is used as intended without modification and only the original user.
- Do not reuse on another prosthesis
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



1218 IFU REV. A(4-30-2026)
Specifications subject to change.