



Double Ended Adapter with Pyramid

Instructions For Use

1261, 1262, 1263, & 1264



1261



1262



1263



1264

1. Application & Description

- This adapter is exclusively for lower extremity prosthetic fittings.
- It connects any pyramid and receiver system.
- The four set screws anchor the mating system and allow for angular alignment.

2. Technical Specifications

P/N	Material	Build Height	Weight	Weight Limit
1261S	Stainless Steel	32mm	121g	136kg/300lbs
1261T	Titanium	32mm	75g	170kg/375lbs
1262S	Stainless Steel	45mm	146g	136kg/300lbs
1262T	Titanium	45mm	86g	170kg/375lbs
1263S	Stainless Steel	60mm	166g	136kg/300lbs
1263T	Titanium	60mm	99g	170kg/375lbs
1264S	Stainless Steel	75mm	188g	136kg/300lbs
1264T	Titanium	75mm	111g	170kg/375lbs

- **Recommended Torque:** Always cross tighten Alignment Set Screws
- **M8 Alignment Set Screw:** 10-15 Nm
- **Thread Locker:** Apply Loctite once alignment and connection are finalized.

3. Safety & Environment

- **Temperature Range:** Between -20°C to 80°C (-4°F to 176°F)
- Rinse with clean water if exposed to salty, chlorinated, or otherwise potentially corrosive liquids
- **Storage:** Store at room temperature; avoid wet, very hot, or extremely cold environments
- **Warranty:** 2 years, provided it is used as intended without modification, and only for the original intended user
- Do not reuse on another prosthesis.
- **Conformity:** This product meets the requirements Council EU 2017/745 (MDR) for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. Please keep this manual in safe place for future reference.



1261_1262_1263_1264 IFU REV.B (5-12-2026)

Specifications subject to change.